# AUTOSTAT™II ACA Screen

## Anti-Cardiolipin Screen

**REF** FGA18

### PROPRIETARY AND COMMON NAMES

Hycor Biomedical Autostat™II Anti-Cardiolipin enzyme linked immunosorbent assay (ELISA) specific for antibodies to cardiolipin.

### INTENDED USE

Enzyme-linked immunosorbent assay method for the detection of IgG, IgM and IgA autoantibodies to cardiolipin in human serum. The results of the ACA assay can be used as an aid in the diagnosis of auto-immune diseases associated with elevated levels of anti-cardiolipin antibodies including anti-phospholipid syndrome. Levels of these autoantibodies are one indicator in a multi-factorial diagnostic regime.

This device has been validated for use with the Hycor HYTEC automated EIA instrument. In addition, a manual assay protocol is described below.

For *in vitro* diagnostic use only.

### INTRODUCTION

Anti-cardiolipin and anti-phosphatidyl serine antibodies along with others, such as the Lupus Anticoagulant (LAC), belong to the family of anti-phospholipid antibodies. Anti-cardiolipin antibodies are circulating serum antibodies often associated with recurrent arterial and venous thromboembolism, recurrent foetal loss and thrombocytopenia.

These symptoms are often present in cases of Systemic Lupus Erythematosus (SLE) and in many other conditions, both of an autoimmune and of a non-autoimmune nature. Some studies show that over 50% of SLE patients have one or more classes of anti-cardiolipin antibodies. The presence of this antibody serves as a marker for the risk of a thromboembolic event. Those SLE patients exhibiting high levels of these autoantibodies are 4 times more likely to have such an event than those not expressing the autoantibodies.

Anti-cardiolipin autoantibodies can be of any combination of the IgM, IgA and IgG classes. IgG antibodies are the most prevalent class of autoantibody and the class with the greatest clinical correlation. Samples found to have IgG levels in the 'high' anti-cardiolipin band are the most likely to display overt clinical symptoms. However, IgA and IgM autoantibodies are often found either alone or in association with the IgG class. Measurement of all three classes is therefore recommended.

Anti-cardiolipin are the most commonly measured anti-phospholipid antibodies. However, it is known that some patients with infectious diseases and other autoimmune disorders show some antibody activity against cardiolipin.


### PRINCIPLE OF THE PROCEDURE

The Autostat™II assay for detection of autoantibodies is a solid phase immunosorbent assay (ELISA) in which the analyte is indicated by a colour reaction of an enzyme and substrate. The Autostat™II wells are coated with purified antigen (1).

On adding diluted serum to the wells the antibodies (2) present bind to the antigen. After incubating at room temperature and washing away unbound material, horseradish peroxidase conjugated to a polyclonal antibody to human polyvalent immunoglobulins (IgG, IgM and IgA) (3) is added, which binds to the immobilised antibodies.

Following further incubation and washing, tetra-methyl benzidine substrate (TMB) (4) is added to each well. The presence of the antigen-antibody-conjugate complex turns the substrate to a dark blue colour. Addition of the stop solution turns the colour to yellow.

The colour intensity is proportional to the amount of autoantibodies present in the original serum sample.

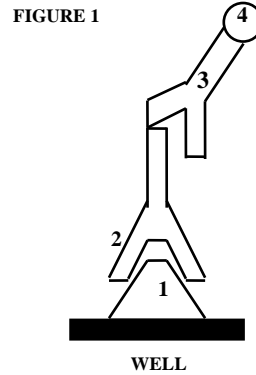COLORLESS  
↓  
BLUE  
ADD STOP SOLUTION  
↓  
YELLOW

- |                     |
|---------------------|
| 1. Antigen          |
| 2. Analyte          |
| 3. Enzyme Conjugate |
| 4. Substrate        |

### MATERIALS PROVIDED

- Goat anti-human IgG, IgM, IgA conjugate: One vial containing 15ml of ready-to-use HRP conjugate. Conjugate diluent contains 0.05% Proclin 300. Conjugates are color coded pink.
- Microplate: One microplate is supplied which contains 12 strips of 8 breakpart wells. The wells are coated with purified cardiolipin from bovine heart.
- Standards: One vial containing 1ml of standard. This is calibrated in arbitrary units and the value allocated is 60 u/ml. The standard contains 0.09% sodium azide as a preservative.
- Positive control: One vial containing 0.45 ml of concentrated positive control which contains human antisera and 0.09% sodium azide as a preservative.
- Negative control: One vial containing 0.45 ml of concentrated negative control which contains normal human serum and 0.09% sodium azide as a preservative.
- TMB substrate solution: One vial containing 15ml of ready-to-use tetra-methylbenzidine (TMB) substrate.
- Sample diluent: One bottle containing 100ml of ready-to-use sample diluent buffer. The buffer includes 0.09% Sodium azide. Sample diluent buffer is colour coded blue.
- Wash buffer: One bottle containing 50ml of wash buffer concentrate. Wash buffer concentrate contains 0.06% Proclin 300.
- Stop solution: One bottle containing 20ml of 0.25M H<sub>2</sub>SO<sub>4</sub> stop solution.

### WARNINGS OR PRECAUTIONS

Warning - Potentially Hazardous and Biohazardous Materials

Sera used in the preparation of the standards and controls have been tested for the presence of antibodies to Human Immunodeficiency Virus (HIV 1 and 2), as well as for Hepatitis B Surface Antigen (HBsAg) and HCV and found to be negative. All material is tested with FDA approved assays. Because no test method can offer complete assurance that HIV, HBsAg or other infectious agents are absent, it is recommended that human serum based products be handled with the same precautions used for patient specimens.

Dispose of reagent solutions containing sodium azide and thimerosal as preservatives according to all local, state and national regulations. To dispose of reagents containing sodium azide, flush away using copious amounts of water. Dispose with caution as sodium azide can form explosive compounds on prolonged contact with lead or copper piping.

The performance data represented here were obtained using specific reagents listed in the package insert. Do not use reagents from other manufacturers in the kits. Do not dilute or adulterate the kit reagents, unless directed by the kit protocol. Do not use the substrate solution if it has begun to turn blue.

Do not use heat-inactivated serum.

Reagents contain preservatives which may be toxic if ingested. Do not pipette by mouth. Avoid contact of reagents or patient samples with skin or mucous membranes. If contact occurs, immediately flush with large quantities of water. Avoid splashing or creation of aerosols. Reusable glassware must be thoroughly washed and rinsed so that it is free of all detergents.

Microplate washing is important. Improperly washed wells will give erroneous results. Do not allow the wells to dry between incubations. Do not vary reagents and incubation temperatures above or below room temperature (18 - 25°C).

## MATERIALS REQUIRED BUT NOT SUPPLIED

The items listed below may not all be required if using the HYTEC assay system. However, if using the HYTEC assay system additional reagents may be required. Support reagents are available from your supplier.

1. Distilled or deionized water.
2. Wash bottle, automated or semi-automated microwell plate washing system.
3. Rack for sample dilution.
4. Micropipettes including multichannels capable of accurately delivering 5-1000µl (less than 3% cv).
5. Reagent reservoirs for multichannel pipettes.
6. One-liter graduated cylinder.
7. Disposal basins and 0.5% sodium hypochlorite (50 ml bleach in 950ml water).
8. Single or dual wavelength microplate reader with 450nm filter. If dual wavelength is used, set the reference filter to 600 - 650 nm.
9. Paper towels, pipette tips and timer

## STORAGE AND SHELF LIFE

Store kit components at 2-8°C and do not use after the expiry date on the box outer label. Before use all components should be allowed to warm up to ambient temperature (18-25°C). After use, the plate should be resealed, the bottle caps replaced and tightened and the kit stored at 2-8°C. The opened kit should be used within three months.

## INTERFERENCES

Grossly haemolysed, lipaemic or microbiologically contaminated samples should not be used. Samples with abnormally elevated levels of haemoglobin, bilirubin and especially EDTA may interfere with assay performance and accuracy.

## SPECIMEN COLLECTION AND PREPARATION

Use serum in this procedure. It is most important to preserve the chemical integrity of a blood specimen from the moment it is collected until it is assayed. Obtain patient samples by non-traumatic venipuncture, using a vacuum tube or sterile syringe. If a syringe is used, transfer the blood immediately to a vacuum tube (plain red-top or serum separator).

Allow samples to clot at room temperature (18-25°C) for at least 20-30 minutes, until the clot just begins to retract. Spin the sample in a centrifuge. Following the centrifugation, transfer the cell-free serum to a tightly stoppered storage tube.

Do not use sera samples showing signs of haemolysis. If it is necessary to store a sample prior to analysis, it is recommended that, for a period of up to 72 hours, store the sample in a sealed container at 2-8°C. Freeze samples at -20°C if longer storage is required. Avoid repeat freeze-thawing.

## LIMITATIONS OF USE

A negative result should not be used as a sole criterion to rule out anti-phospholipid syndrome or other autoimmune disease, but must be taken in relation to other clinical observations and diagnostic tests. While the precision of the Autostat™II kit is sufficient to allow samples to be measured in single determinations, this is done at the clinical laboratory's discretion. It is advised that duplicate determinations should be used to enable identification of potential pipetting error or to allow for confirmation in the equivocal range.

It should be noted that anti-cardiolipin antibodies occur at low levels in other autoimmune and non-autoimmune conditions. Therefore all other clinical observations and diagnostic tests should be taken into account for clinical diagnosis.

## REAGENT PREPARATION

- Wash buffer: Measure and dilute 50 ml of the 20 X wash buffer to one liter with distilled or deionized water. Mix well before use. Store this solution at 2-8°C if it is not to be used immediately. The diluted wash buffer is stable at 2-8°C for one week.

**Important note: Do not use wash buffer from other Autostat™II kits as this will affect the assay and produce unreliable results.**

- Positive and negative controls: These are provided in a concentrated form and should be diluted 1/100 with sample diluent buffer before use. Prepare fresh control dilutions before each assay run. Vortex all samples and controls before testing.

The conjugate, substrate and stop solution in Autostat™II kits are supplied in a ready-to-use format.

## MANUAL ASSAY PROTOCOL

1. Bring all reagents to room temperature (18-25°C).
2. Select sufficient microwells for the test. Remove protective covering and select sufficient wells to accommodate the patient samples, standards and assay controls. Each sample is recommended to be tested in duplicate.
3. Dilute all serum samples and assay controls 1/100 in sample diluent by adding 5µl to 495µl sample diluent. Standards do not require dilution.
4. Pipette 100µl of the standards, diluted control or diluted patient sample into the wells. To achieve blanking on the plate reader add a 'no serum' control of 100µl of sample diluent to the first two wells. This will act as the zero point for the curve fit.
5. Incubate the wells at room temperature (18-25°C) for 30 minutes.
6. Wash the wells three times with diluted wash buffer. This can be done manually with a multichannel pipette or on an automatic plate washer. Empty the wells, invert and tap dry on paper towel.
7. Add 100µl of ready-to-use conjugate to each well.
8. Incubate the wells at room temperature (18-25°C) for 15 minutes.
9. Repeat washing as in section 6. above.
10. Add 100µl of ready-to-use TMB substrate to each well.
11. Incubate the wells at room temperature (18-25°C) for 15 minutes.
12. Add 50µl of stop solution to each well. Tap gently to ensure uniform color distribution and read within 15 minutes.
13. To read the plate, ensure the base is free from moisture and no air bubbles are in the wells. Read the absorbance of the well contents at 450nm on a suitable plate reader. On readers equipped with a dual wavelength facility set the reference filter to 600 - 650nm.
14. Subtract the blank (or mean of blanks) from the optical densities of the standard, controls and patient samples. If the assay was performed in duplicate, the mean of the wells should be taken.

## HYTEC INSTRUMENT ASSAY PROTOCOL

Please refer to the HYTEC automated EIA system procedure manual for specific instructions on using this assay with the HYTEC automated instrument. Dilutions of controls and samples are performed automatically by the HYTEC instrument. Assay results are calculated by the HYTEC system automatically.

## CALCULATION OF RESULTS

### Qualitative Calculation:

Calculate the mean, blank corrected, absorbance value (OD) for duplicates of the kit standard. Calculate the mean, blank corrected, absorbance value (OD) for duplicates of the kit controls and patient samples. Using the following algorithm, calculate the concentration of each of the samples:

$$\frac{\text{Concentration of standard} \times \text{OD of sample or control}}{\text{OD of standard}}$$

The concentration of the standard is 60 u/ml.

**TYPICAL RESULTS**

The data below are for example only and should not be used for calculation of results. The results were calculated using the formula described in the 'Calculation of Results' section.

	OD	OD	mean OD	Result
Reagent Blank	0.083	0.074	0.079	
Standard	2.336	2.383	2.359	
Sample 1	1.462	1.630	1.546	38.6 u/ml
Sample 2	0.349	0.314	0.332	6.6 u/ml

**INTERPRETATION OF RESULTS**

A total of 116 samples were assayed. 67 of the samples were from normal subjects and 49 from people with an autoimmune condition. The cut-offs shown below were based on 67 normal samples tested.

u/ml	Negative	Positive
ACA screen	< 10	≥ 10

It is recommended that samples giving a positive result be retested for specific levels of IgG, IgM and IgA anti-cardiolipin antibodies.

**PERFORMANCE CHARACTERISTICS**

**Analytical Specificity and Sensitivity**

Of a panel of 67 'normal' asymptomatic individuals four samples gave a positive result. Three of these positive samples gave a positive ACA IgG result. Of a panel of 49 potentially ACA positive samples, 45 (91.8%) were positive. Of a panel of 48 samples from people with a suspected autoimmune condition, 19 tested positive in the ACA Screen assay. 17 of these samples were verified as positive for IgG, IgM or IgA ACA, the remaining two were at the cut-off.

The sensitivity of the assay was established by calculation of the mean plus two standard deviations of 22 replicates of the zero standard which gave a value of 1.03 u/ml.

**Relative Specificity and Sensitivity**

The assay was compared with another commercially available test. The results are shown below:

		Autostat II		Relative sensitivity	100%
		Positive	Negative		
Competitor	Positive	22	0	Relative specificity	95%
	Negative	2	38	Overall agreement	96.8%

**Reproducibility**

Inter-assay variation	A	B	C
Mean of duplicates	6.6	19.0	55.1
%CV	4.9	2.4	1.3

Intra-assay variation	A	B	C
Mean of duplicates	14.3	19.4	57.7
%CV	9.7	3.5	6.1

**Reactivity Against Samples With Known Autoantibody Specificities.**

Reference preparations were obtained from International Standards Organisations, and tested on the Autostat II ACA screen kit. Results are summarised and discussed below:

SSA	SSB	Sm	RNP	Jo-1	Scl-70	RF
CDC	CDC	CDC	CDC	CDC	CDC	WHO
neg	neg	pos	neg	neg	neg	neg

dsDNA	ANA(nucl)	ANA(speck)	ANA(cent)
WHO	CDC	CDC	CDC
neg	neg	neg	neg

Anti-Sm antibodies are indicative of Systemic Lupus Erythematosus (SLE). Patients with SLE frequently have elevated levels of anti-cardiolipin antibodies.

**QUALITY CONTROL**

Good laboratory practice indicates that with each assay run, one or more quality control samples of known antibody level should be analyzed as though they were clinical samples. Positive and negative control samples are supplied with each kit, which may be assayed with each run. The results of these quality control samples should fall within the limits indicated on the Certificate of Analysis.

Should the results fall outwith this range repeat the assay using freshly prepared controls. Should the results continue to fall outside the specific range, and after equipment, adherence to the protocol and laboratory procedure have been verified, seek assistance from the supplier. Do not report patient results if the control results fall outwith the acceptable ranges.

**Hycor Biomedical**  
 7272 Chapman Ave  
 Garden Grove  
 California 92841  
 U.S.A.  
 +1 800 382 2527 (Tel)  
 +1 714 901 1264 (Fax)  
[www.hycorbiomedical.com](http://www.hycorbiomedical.com)

**Advena Ltd.**  
 Pure Offices  
 Plato Close  
 Warwick CV34 6WE  
 U.K.  
 +44(0) 1926 800153 (Tel)

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## BIBLIOGRAPHY

- Harris EN, Gharavi AE, Patel SP, Hughes GRV (1987) Evaluation of anti-cardiolipin antibody test. Report of an International Workshop, held 4 April 1986. Clin. Exp. Immunol., **68**, 215-222.
- Gharavi AE, Harris EN, Asherson RA, Hughes GRV (1987) Anti-cardiolipin antibodies: isotype distribution and phospholipid specificity. Ann. Rheum. Dis., **46**, 1-6.
- Escalante A, Brey RL, Mitchell Jr BD, Dreiner U (1995) Accuracy of anti-cardiolipin antibodies in identifying a history of thrombosis among patients with Systemic Lupus Erythematosus. Am. J. of Medicine, **98**, 559-565.
- Paggi A, Caccavo D, Ferri GM, et al. (1994) Anti-cardiolipin antibodies in autoimmune thyroid diseases. Clin. Endocrinol., **40**, 329-333.
- Cronin ME, Biswas RM, Van der Straeton C, Fleisher TA, Klippel JH. (1988) IgG and IgM anti-cardiolipin antibodies in patients with lupus with anti-cardiolipin antibody associated syndromes. J. Rheumatol., **15**, 795-798.
- Rose NS, Dostal-Johnson D, Ware Branch MS, Ware Branch D. (1990) Anti-phospholipid antibodies and recurrent pregnancy loss: correlation between the activated partial thromboplastin time and antibodies against phosphatidyl serine and cardiolipin. Am. J. Obstet. Gynecol., **163**, 575-584.

**Hycor Biomedical**  
7272 Chapman Ave  
Garden Grove  
California 92841  
U.S.A.  
+1 800 382 2527 (Tel)  
+1 714 901 1264 (Fax)

www.hycorbiomedical.com

**Advna Ltd.**  
Pure Offices  
Plato Close  
Warwick CV34 6WE  
U.K.  
+44(0) 1926 800153 (Tel)

## Performance Data/Performances/Typische Ergebnisse/Caratteristiche/Datos de actuación.

	Manual	HyTec
Analytical sensitivity/Limite de détection/Analytische Empfindlichkeit/Sensibilità analitica/Sensibilidad analítica	1.03 u/ml	0.7 u/ml
Relative specificity /Spécificité relative /Spezifitat/Specificite/ Especificidad	95%	95.7%
Relative sensitivity/Sensibilit� relative/ Empfindlichkeit / Sensibilit�/ Sensibilidad relativa	100%	90.9%
Intra-assay variation/ la reproductibilit� intra-lot/Intra Testvariationen/Riproducibilit� intralot/Variaci�n intraan�lisis	A 9.7%	A 8.1%
	B 3.5%	B 7.9%
	C 6.1%	C 9.7%
Inter-assay variation/ la reproductibilit� inter-lot/ Intra Testvariationen /Riproducibilit� interlotto/ Variaci�n interan�lisis	A 4.9%	A 7.9%
	B 2.4%	B 4.5%
	C 1.3%	C 7.3%

	English	Français	Deutsch	Italiano	Español
<b>LOT</b>	Batch / Lot code	Code du lot	Chargenbezeichnung	Codice del lotto	Codigo de lote
Expiry / Use by	Expiry / Use by	Date du expiration	Verwendbar bis	Utilizzare entro	Fecha de caducidad
Manufactured by	Manufactured by	Fabriqué par	Hergestellt von	Fabbricato da	Fabricado por
Store at	Store at	Conserver à	Lagerung bei	Conservare a	Conservar a
See instructions for use	See instructions for use	Consulter les instructions d'utilisation	Gebrauchsanweisung beachten	Consultare le istruzioni per l'uso	Consulte las instrucciones de uso
<b>IVD</b>	For in vitro diagnostic use only	Pour utilisation in vitro uniquement	Nur zur in vitro Verwendung	Per uso diagnostico in vitro	S�lo para el uso diagn�stico in vitro
<b>CONTROL-</b>	Negative control	Contr�le n�gatif	Negativ Kontrolle	Controllo negativo	Control negativo
<b>CONTROL+</b>	Positive control	Contr�le positif	Positiv Kontrolle	Controllo positivo	Control positivo
<b>REF</b>	Catalogue number	Code produit	Bestellnummer	Numero di catalogo	N�mero de cat�logo
<b>ECIREP</b>	Authorized representative	Repr�sentant Autoris�	Autorisierter Repr�sentant	Rappresentante Autorizzato	Representante Autorizado
$\Sigma$	Contains sufficient for <n> tests	Contenu suffisant pour <n> tests	Enth�lt ausreichend Reagenzien f�r <n> Tests	Contenuto sufficiente per <n>test	Contiene lo necesario para <n> test
<b>CAL</b>	Calibrator	Calibrateur	Kalibrator	Calibratore	Calibrador
<b>CONJ</b>	Conjugate	Conjug�	Konjugat	Conjugate	Conjugar
<b>MT PL ATF</b>	Microtiter plate	Plaque de microtitration	Mikrotiterplatte	Piatto Microtiter	Plato Microtiter
<b>SAMPLEDIL</b>	Sample diluent	Diluant	Proben verd�nner	Diluire prova	Diluir muestra
<b>BUF WASH 20X</b>	Wash buffer Concentrate	Solution de lavage Concentr�	Waschpuffer Konzentrat	lavari di tampone concentrare	lavado de defensa concentrar
<b>SUBS</b>	Substrate	Substrat	Substrat	Substrato	Substrato
<b>SOLN STOP</b>	Stop solution	Solution d'arr�t	Stop L�sung	terminare la soluzione	Parar la soluci�n