# AUTOSTAT™II ENA-6 Profile

Extractable Nuclear Antigen-6 Profile

**REF** FGA38

## PROPRIETARY AND COMMON NAMES

Hycor Biomedical Autostat™II ENA-6 Profile enzyme linked immunosorbent assay (ELISA) specific for antibodies to SS-A/Ro, SS-B/La, Sm, Sm/RNP, Jo-1 and Scl-70.

## INTENDED USE

Enzyme-linked immunosorbent assay method for the simultaneous semi-quantitative determination of specific IgG autoantibodies to SS-A/Ro, SS-B/La, Sm, Sm/RNP, Jo-1 and Scl-70 in human serum. The results of the anti-ENA 6 Profile assays can be used as an aid in the diagnosis of auto-immune diseases including Systemic Lupus Erythematosus (SLE), Sjögren's Syndrome (SS), Mixed Connective Tissue Disease (MCTD), Poly/Dermato myositis (PM/DM) and Scleroderma. Levels of these autoantibodies are one indicator in a multi-factorial diagnostic regime.

For *in vitro* diagnostic use only.

## INTRODUCTION

The detection of Anti-Nuclear Antibodies (ANA's) has long been an important tool in the diagnosis of systemic rheumatic diseases. The antigens used in their detection are purified by the saline extraction of human or animal nuclei, this has led to them being termed Extractable Nuclear Antigens (ENA's). The most commonly measured ENA specifications are anti-SS-A/Ro, anti-SS-B/La, anti-Sm, anti-Sm/RNP, anti-Jo-1 and anti-Scl-70.

The intracellular antigens SS-A/Ro, SS-B/La, Sm, Sm/RNP, Jo-1 and Scl-70 are targets for autoimmune responses in many patients with rheumatic diseases. Anti-SS-A/Ro antibodies are found in 30-50% of patients with Systemic Lupus Erythematosus (SLE), but most significantly in around 95% of patients with primary or secondary Sjögren's Syndrome (SS). Anti-SS-B/La antibodies are also found in SLE and SS patients. Anti-Sm antibodies are considered highly specific for SLE and approximately 30-40% of patients show their presence. Anti-RNP antibodies are predominantly found in patients with Mixed Connective Tissue Disease (MCTD) but are also associated with SLE, SS and Scleroderma. Anti-Jo-1 antibodies are considered specific for Polymyositis and Dermatomyositis. Anti-Scl-70 antibodies are recognised as specific markers for Primary Systemic Sclerosis (PSS or Scleroderma).

Autoantibodies to ENA's vary depending on disease state. The following are estimated incidences of ENA antibodies in various diseases:

Antibody	SLE	SS	PSS	MCTD	PM	DM
SS-A/Ro	30-50%	~95%	---	---	---	---
SS-B/La	>15%	~87%	---	---	---	---
Sm	30-40%	---	---	---	---	---
Sm/RNP	35-45%	>30%	>20%	95-100%	---	---
Jo-1	---	---	---	---	>25%	>25%
Scl-70	---	---	20-30%	---	---	---

Many methods are used for the detection of anti-nuclear antibodies including immunofluorescence and double immunodiffusion Ouchterlony assay, however, ELISA is rapidly becoming the method of choice.

## PRINCIPLE OF THE PROCEDURE


The Autostat™II assay for detection of autoantibodies is a solid phase immunosorbent assay (ELISA) in which the analyte is indicated by a colour reaction of an enzyme and substrate. The Autostat™II wells are coated with purified antigen (1).

On adding diluted serum to the wells the antibodies (2) present bind to the antigen. After incubating at room temperature and washing away unbound material, horseradish peroxidase conjugated anti-IgG monoclonal antibody (3) is added, which binds to the immobilised antibodies.

Following further incubation and washing, tetra-methyl benzidine substrate (TMB) (4) is added to each well. The presence of the antigen-antibody-conjugate complex turns the substrate to a dark blue colour. Addition of the stop solution turns the colour to yellow.

The colour intensity is proportional to the amount of autoantibodies present in the original serum sample.

FIGURE 1

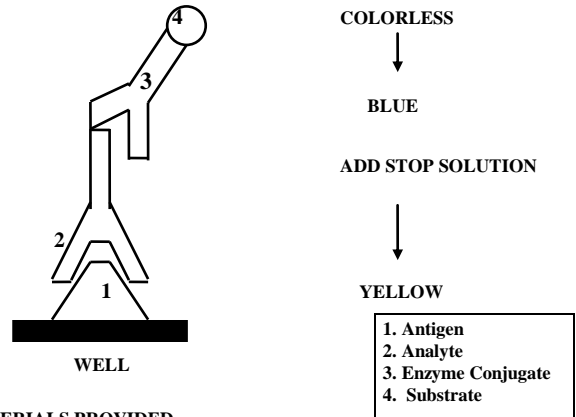## MATERIALS PROVIDED

- Mouse anti-human IgG conjugate: One vial containing 15ml of ready-to-use HRP conjugate. Conjugate diluent contains 0.05% Proclin 300. Conjugates are color coded pink.

- Microplate: One microplate is supplied which contains 16 wells of each antigen in a breakapart format. The wells are coated with purified ENAs of bovine origin, and are colour-coded according to the antigen:

SS-A	yellow	Sm/RNP	red
SS-B	light blue	Jo-1	white
Sm	green	Scl-70	black

- Standards: Six vials each containing 1ml of standard. Each standard represents 100u/ml for that parameter. The standards are in arbitrary units and contain human antisera, and 0.09% sodium azide as a preservative. The following are included: Anti-SS-A/Ro, Anti-SS-B/La, Anti-Sm, Anti-Sm/RNP, Anti-Jo-1 and Anti-Scl-70.

- Positive controls: Six vials each containing 0.45 ml of concentrated positive control which contains human antisera and 0.09% sodium azide as a preservative. There is one positive control for each parameter. Each is colour-coded to match the coated wells.

- Negative control: One vial containing 0.45 ml of concentrated negative control which contains normal human serum and 0.09% sodium azide as a preservative. This can be used for each of the parameters.

- TMB substrate solution: One vial containing 15ml of ready-to-use tetra-methylbenzidine (TMB) substrate.

- Sample diluent: One bottle containing 100 ml of ready-to-use sample diluent buffer. The buffer includes 0.09% Sodium azide. Sample diluent buffer is color coded blue.

- Wash buffer: One bottle containing 50ml of wash buffer concentrate. Wash buffer concentrate contains 0.06% Proclin 300.

- Stop solution: One bottle containing 20ml of 0.25M H<sub>2</sub>SO<sub>4</sub> stop solution.

## WARNINGS OR PRECAUTIONS

Warning - Potentially Hazardous and Biohazardous Materials

Sera used in the preparation of the standards and controls have been tested for the presence of antibodies to Human Immunodeficiency Virus (HIV 1 and 2), as well as for Hepatitis B Surface Antigen (HBsAg) and HCV and found to be negative. All material is tested with FDA approved assays. Because no test method can offer complete assurance that HIV, HBsAg or other infectious agents are absent, it is recommended that human serum based products be handled with the same precautions used for patient specimens.

Dispose of reagent solutions containing sodium azide and thimerosal as preservatives according to all local, state and national regulations. To dispose of reagents containing azide, flush away using copious amounts of water. Dispose with caution as azide can form explosive compounds on prolonged contact with lead or copper piping.

The performance data represented here were obtained using specific reagents listed in the package insert. Do not use reagents from other manufacturers in the kits. Do not dilute or adulterate the kit reagents, unless directed by the kit protocol. Do not use the substrate solution if it has begun to turn blue. Do not use heat-inactivated serum.

Reagents contain preservatives which may be toxic if ingested. Do not pipette by mouth. Avoid contact of reagents or patient samples with skin or mucous membranes. If contact occurs, immediately flush with large quantities of water. Avoid splashing or creation of aerosols. Reusable glassware must be thoroughly washed and rinsed so that it is free of all detergents.

Microplate washing is important. Improperly washed wells will give erroneous results. Do not allow the wells to dry between incubations. Do not vary reagents and incubation temperatures above or below room temperature (18 - 25°C).

## MATERIALS REQUIRED BUT NOT SUPPLIED

The items listed below may not all be required if using the HYTEC assay system. However, if using the HYTEC assay system additional reagents may be required. Support reagents are available from your supplier.

1. Distilled or deionized water.
2. Wash bottle, automated or semi-automated microwell plate washing system.
3. Rack for sample dilution.
4. Micropipettes including multichannels capable of accurately delivering 5-1000µl (less than 3% cv).
5. Reagent reservoirs for multichannel pipettes.
6. One-liter graduated cylinder.
7. Disposal basins and 0.5% sodium hypochlorite (50 ml bleach in 950ml water).
8. Single or dual wavelength microplate reader with 450nm filter. If dual wavelength is used, set the reference filter to 600 - 650 nm.
9. Paper towels, pipette tips and timer

## STORAGE AND SHELF LIFE

Store kit components at 2-8°C and do not use after the expiry date on the box outer label. Before use all components should be allowed to warm up to ambient temperature (18-25°C). After use, the plate should be resealed, the bottle caps replaced and tightened and the kit stored at 2-8°C. The opened kit should be used within three months.

## INTERFERENCES AND HOOK EFFECT

Grossly haemolysed, lipaemic or microbiologically contaminated samples should not be used. Samples with abnormally elevated levels of haemoglobin, bilirubin and especially EDTA may interfere with assay performance and accuracy.

A 'hook effect' may only be seen with very high samples which are above the assay range. Since this is a semi-quantitative assay, any hook effect above the assay range will not influence the performance or accuracy. This is a sample specific effect.

## SPECIMEN COLLECTION AND PREPARATION

Use serum in this procedure. It is most important to preserve the chemical integrity of a blood specimen from the moment it is collected until it is assayed. Obtain patient samples by non-traumatic venipuncture, using a vacuum tube or sterile syringe. If a syringe is used, transfer the blood immediately to a vacuum tube (plain red-top or serum separator).

Allow samples to clot at room temperature (18-25°C) for at least 20-30 minutes, until the clot just begins to retract. Spin the sample in a centrifuge. Following the centrifugation, transfer the cell-free serum to a tightly stoppered storage tube.

Do not use sera samples showing signs of haemolysis. If it is necessary to store a sample prior to analysis, it is recommended that, for a period of up to 72 hours, store the sample in a sealed container at 2-8°C. Freeze samples at -20°C if longer storage is required. Avoid repeat freeze-thawing.

## LIMITATIONS OF USE

A negative result should not be used as a sole criterion to rule out connective tissue disease or other autoimmune disease, but must be taken in relation to other clinical observations and diagnostic tests. While the precision of the Autostat™II kit is sufficient to allow samples to be measured in single determinations, this is done at the clinical laboratory's discretion. It is advised that duplicate determinations should be used to enable identification of potential pipetting error or to allow for confirmation in the equivocal range.

It should be noted that ENA antibodies occur at low levels in other autoimmune and non-autoimmune conditions. Therefore all other clinical observations and diagnostic tests should be taken into account for clinical diagnosis.

## REAGENT PREPARATION

- Wash buffer: Measure and dilute 50 ml of the 20 X wash buffer to one liter with distilled or deionized water. Mix well before use. Store this solution at 2-8°C if it is not to be used immediately. The diluted wash buffer is stable at 2-8°C for one week.

- Positive and negative controls: These are provided in a concentrated form and should be diluted 1/100 with sample diluent buffer before use. Prepare fresh control dilutions before each assay run. Vortex all samples and controls before testing.

The conjugate, substrate and stop solution in Autostat™II kits are supplied in a ready-to-use format.

## MANUAL ASSAY PROTOCOL

1. Bring all reagents to room temperature (18-25°C).
2. Select sufficient microwells for the test. Remove protective covering and select sufficient wells to accommodate the patient samples, standards and assay controls. Each sample is recommended to be tested in duplicate.
3. Dilute all serum samples and assay controls 1/100 in sample diluent by adding 5µl to 495µl sample diluent. Standards do not require dilution.
4. Pipette 100µl of the standards, diluted control or diluted patient sample into the wells. To achieve blanking on the plate reader add a 'no serum' control of 100µl of sample diluent to the first two wells. This will act as the zero point for the curve fit.
5. Incubate the wells at room temperature (18-25°C) for 30 minutes.
6. Wash the wells three times with diluted wash buffer. This can be done manually with a multichannel pipette or on an automatic plate washer. Empty the wells, invert and tap dry on paper towel.
7. Add 100µl of ready-to-use conjugate to each well.
8. Incubate the wells at room temperature (18-25°C) for 15 minutes.
9. Repeat washing as in section 6. above.
10. Add 100µl of ready-to-use TMB substrate to each well.
11. Incubate the wells at room temperature (18-25°C) for 15 minutes.
12. Add 50µl of stop solution to each well. Tap gently to ensure uniform color distribution and read within 15 minutes.
13. To read the plate, ensure the base is free from moisture and no air bubbles are in the wells. Read the absorbance of the well contents at 450nm on a suitable plate reader. On readers equipped with a dual wavelength facility set the reference filter to 600 - 650nm.
14. Subtract the blank (or mean of blanks) from the optical densities of the standard, controls and patient samples. If the assay was performed in duplicate, the mean of the wells should be taken.

## CALCULATION OF RESULTS

Calculate the mean, blank corrected, absorbance value (OD) for duplicates of the kit standard. Calculate the mean, blank corrected, absorbance value (OD) for duplicates of the kit controls and patient samples. Using the following algorithm, calculate the concentration of each of the samples:

$$\frac{\text{Concentration of standard}}{\text{OD of standard}} \times \text{OD of sample or control}$$

The concentration of each standard is 100 u/ml.

## MANUAL TYPICAL RESULTS

The data below are for example only and should not be used for calculation of results. The results were calculated using the formula described in the 'Calculation of Results' section.

		OD	OD	Mean OD	Result u/ml
<b>SS-A</b>	Blank	0.197	0.203	0.200	
	Standard	1.954	2.096	2.025	
	Sample 1	2.062	2.107	2.085	103.3
	Sample 2	1.133	1.278	1.206	55.1
<b>SS-B</b>	Blank	0.068	0.069	0.069	
	Standard	1.904	1.923	1.914	
	Sample 1	0.791	0.796	0.794	39.3
	Sample 2	0.510	0.507	0.508	23.8
<b>Sm</b>	Blank	0.055	0.046	0.051	
	Standard	1.835	1.936	1.885	
	Sample 1	1.141	1.198	1.169	61.0
	Sample 2	0.510	0.507	0.508	24.9
<b>Sm/RNP</b>	Blank	0.059	0.059	0.059	
	Standard	2.144	2.082	2.113	
	Sample 1	1.650	1.585	1.618	75.9
	Sample 2	0.995	0.969	0.982	44.9
<b>Jo-1</b>	Blank	0.046	0.048	0.047	
	Standard	1.588	1.681	1.635	
	Sample 1	1.727	1.649	1.688	103.3
	Sample 2	0.889	0.844	0.867	51.6
<b>Scl-70</b>	Blank	0.048	0.049	0.049	
	Standard	2.384	2.529	2.457	
	Sample 1	1.278	1.420	1.349	54.0
	Sample 2	0.789	0.769	0.779	30.3

### INTERPRETATION OF RESULTS

The assay cut-off's were determined by combining data from a panel of samples including normal samples and samples from people with an autoimmune condition. Details are shown in the table below:

Parameter	Total Samples	No. of Normal samples	No. of autoimmune samples	Age range of normals	No samples used in cut-off calculation
SS-A	100	22	78	18 - 60	78
SS-B	123	45	78	18 - 60	105
Sm	120	28	92	18 - 60	98
Sm/RNP	108	27	81	18 - 60	92
Jo-1	173	40	133	18 - 60	40
Scl-70	186	40	146	18 - 60	166

Arbitrary u/ml	Negative	Equivocal	Positive
SS-A/Ro	<10.0	10.0 - 15.0	>15.0
SS-B/La	<10.0	10.0 - 15.0	>15.0
Sm	<10.0	10.0 - 15.0	>15.0
Sm/RNP	<10.0	10.0 - 15.0	>15.0
Jo-1	<10.0	10.0 - 15.0	>15.0
Scl-70	<10.0	10.0 - 15.0	>15.0

It is recommended that equivocal results be retested using a subsequent sample.

The Sm/RNP parameter measures antibodies to both the Sm and RNP antigens. The Sm antibody result should be taken into account when interpreting the result for RNP antibodies.

### PERFORMANCE CHARACTERISTICS

#### Analytical Specificity and Sensitivity

For each parameter, a panel of 'normal' asymptomatic individuals was run (see table above for the sample size). The range of negative results was 97-100%.

No cross-reactivity with anti-dsDNA, anti-Thyroglobulin, anti-Thyroid Peroxidase, Rheumatoid Factor, anti-Gastric Parietal Cell, anti-Cardiolipin or ANCA was seen.

The sensitivity of the assay was established by calculation of the mean plus two standard deviations of 22 replicates of the zero standard which gave the following values:

<b>SS-A/Ro:</b>	0.8 u/ml	<b>Sm/RNP:</b>	0.7 u/ml
<b>SS-B/La:</b>	1.2 u/ml	<b>Jo-1:</b>	0.5 u/ml
<b>Sm:</b>	0.7 u/ml	<b>Scl-70:</b>	0.6 u/ml

### Relative Specificity and Sensitivity

Each assay was compared to another commercially available test and was found to be substantially equivalent. The results are shown below. Equivocal results were omitted from the calculations.

#### SS-A

Predicate	Positive	Autostat II		
		Equivocal	Negative	
Positive	18	0	0	Relative sensitivity = 100.0 %
Equivocal	3	0	0	Relative specificity = 98.7 %
Negative	1	0	78	Overall agreement = 99.0 %

#### SS-B

Predicate	Positive	Autostat II		
		Equivocal	Negative	
Positive	16	1	1	Relative sensitivity = 94.1 %
Equivocal	0	0	0	Relative specificity = 100 %
Negative	0	0	105	Overall agreement = 99.2 %

#### Sm

Predicate	Positive	Autostat II		
		Equivocal	Negative	
Positive	11	3	0	Relative sensitivity = 100.0 %
Equivocal	1	1	1	Relative specificity = 99.0 %
Negative	1	1	101	Overall agreement = 99.1 %

#### Sm/RNP

Predicate	Positive	Autostat II		
		Equivocal	Negative	
Positive	14	1	1	Relative sensitivity = 93.3 %
Equivocal	0	0	0	Relative specificity = 97.8 %
Negative	2	2	88	Overall agreement = 97.1 %

#### Jo-1

Predicate	Positive	Autostat II		
		Equivocal	Negative	
Positive	22	2	3	Relative sensitivity = 88.0 %
Equivocal	0	1	2	Relative specificity = 99.3 %
Negative	1	2	140	Overall agreement = 97.6 %

#### Scl-70

Predicate	Positive	Autostat II		
		Equivocal	Negative	
Positive	25	2	4	Relative sensitivity = 86.2 %
Equivocal	0	0	1	Relative specificity = 99.3 %
Negative	1	2	151	Overall agreement = 97.2 %

### Reproducibility

Intra- and inter-assay variation were checked using a number of samples:

		Intra-assay variation			Inter-assay variation		
		A	B	C	A	B	C
SS-A	mean	8.6	34.3	64.7	8.6	34.3	64.7
	% CV	6.3	1.9	1.1	8.7	2.3	3.1
SS-B	mean	14.3	24.5	88.7	14.3	24.5	88.7
	% CV	7.7	6.2	7.5	12.3	7.8	6.7
Sm	mean	11.2	41.3	64.5	11.2	41.3	64.5
	% CV	5.4	6.8	2.3	3.7	3.6	4.5
Sm/RNP	mean	14.3	28.1	46.6	14.3	28.1	46.6
	% CV	4.2	2.7	4.6	5.6	5.5	7.1
Jo-1	mean	9.8	18.0	44.4	9.8	18.0	44.4
	% CV	6.5	5.6	4.3	2.8	3.2	4.3
Scl-70	mean	10.0	37.5	53.3	10.0	37.5	53.3
	% CV	2.1	1.7	2.2	3.4	3.7	3.1

### QUALITY CONTROL

Good laboratory practice indicates that with each assay run, one or more quality control samples of known antibody level should be analysed as though they were clinical samples. Cogent Diagnostics supply with each Autostat™II ENA-6 Profile kit, positive and negative control samples which may be assayed with each run. The results of these quality control samples should fall within the limits indicated on the Certificate of Analysis.

Should the results fall outwith this range the assay should be repeated using freshly prepared controls. Should the results continue to fall outside the specific range, and after equipment, adherence to the protocol and laboratory procedure have been verified, assistance from the supplier should be sought. Do not report patient results if the control results fall outwith the acceptable ranges.

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### BIBLIOGRAPHY

- Wilson M, Nitsche J. (1986) Immunodiffusion assays for antibodies to nonhistone nuclear antigens. In: Rose N. (ed) Manual of Clinical Laboratory Immunology, 3rd edition. pp 750-753.
- Ben-Chetrit E, Fox R, Tan E. (1990) Dissociation of immune responses to the SS-A(Ro) 52kD and 60kD polypeptides in Systemic Lupus Erythematosus and Sjogren's Syndrome. Arthritis Rheum, 33, 349-355.
- Pruijn G. (1994) The La(SS-B) antigen. Manuals of Biological Markers of Disease. B4.2, pp 1-14.
- Field M, Williams DG, Charles P, Maini RN (1988) Specificity of anti-Sm antibodies by ELISA for Systemic Lupus Erythematosus: increased sensitivity of detection using purified peptide antigens. Ann. Rheum. Dis., 47, 820-825.
- Klein Gunnewiek JMT, Van Venrooij WJ. (1994) Autoantigens contained in the U-1 small nuclear ribonucleoprotein complex. Manual of Biological Markers of Disease. B3.1, 1-20.
- Walker EJ, Tymms KE, Webb J, Jeffery PD. (1987) Improved detection of anti-Jo-1 antibody, a marker for myositis using purified histidyl-tRNA synthetase. J. Immunol. Methods, 96, 149-156.
- Hildebrandt S, Weiner ES, Senecal J-L, Noell GS, Earnshaw WC, Rothfield NF. (1990) Autoantibodies to topoisomerase I (Scl-70): analysis by gel diffusion, immunoblot and enzyme-linked immunosorbent assay (ELISA). Clin. Immunol. Immunopathol., 57, 399 - 410.

**Hycor Biomedical**  
7272 Chapman Ave  
Garden Grove  
California 92841  
U.S.A.  
+1 800 382 2527 (Tel)  
+1 714 901 1264 (Fax)  
[www.hycorbiomedical.com](http://www.hycorbiomedical.com)

**Advena Ltd.**  
Pure Offices  
Plato Close  
Warwick CV34 6WE  
U.K.  
+44(0) 1926 800153 (Tel)

### Performance Data/Performances/Typische Ergebnisse/Caratteristiche/Datos de actuación.

	Manual
Analytical sensitivity/Limite de détection/Analytische Empfindlichkeit/Sensibilità analitica/Sensibilidad analítica	SS-A 0.8 u/ml; SS-B 1.2 u/ml; Sm 0.7 u/ml; Sm/RNP 0.7 u/ml Jo-1 0.5 u/ml; Scl-70 0.6 u/ml
Relative specificity/Spécificité relative/Spzifitat/Specificite/ Especificidad	SS-A 98.7%; SS-B 100%; Sm 99.0%; Sm/RNP 97.8% Jo-1 99.3%; Scl-70 99.3%
Relative sensitivity/Sensibilité relative/ Empfindlichkeit / Sensibilità/ Sensibilidad relativa	SS-A 100%; SS-B 94.1%; Sm 100%; Sm/RNP 93.3% Jo-1 88%; Scl-70 86.2%
Intra-assay variation/ la reproductibilité intra-lot/Intra Testvariationen/Riproducibilità intralot/Variación intraanálisis	SS-A <10%; SS-B <10%; Sm <10%; Sm/RNP <10% Jo-1 <10%; Scl-70 <10%
Inter-assay variation/ la reproductibilité inter-lot/ Intra Testvariationen /Riproducibilità interlotto/ Variación interanálisis	SS-A <10%; SS-B <13%; Sm <10%; Sm/RNP <10% Jo-1 <10%; Scl-70 <10%

	English	Français	Deutsch	Italiano	Español
<b>LOT</b>	Batch / Lot code	Code du lot	Chargenbezeichnung	Codice del lotto	Código de lote
Expiry / Use by	Expiry / Use by	Date du expiration	Verwendbar bis	Utilizzare entro	Fecha de caducidad
Manufactured by	Manufactured by	Fabriqué par	Hergestellt von	Fabbricato da	Fabricado por
Store at	Store at	Conserver à	Lagerung bei	Conservare a	Conservar a
See instructions for use	See instructions for use	Consulter les instructions d'utilisation	Gebrauchsanweisung beachten	Consultare le istruzioni per l'uso	Consulte las instrucciones de uso
<b>IVD</b>	For in vitro diagnostic use only	Pour utilisation in vitro uniquement	Nur zur in vitro Verwendung	Per uso diagnostico in vitro	Sólo para el uso diagnóstico in vitro
<b>CONTROL -</b>	Negative control	Contrôle négatif	Negativ Kontrolle	Controllo negativo	Control negativo
<b>CONTROL +</b>	Positive control	Contrôle positif	Positiv Kontrolle	Controllo positivo	Control positivo
<b>REF</b>	Catalogue number	Code produit	Bestellnummer	Numero di catalogo	Número de catálogo
<b>ECIREP</b>	Authorized representative	Représentant Autorisé	Autorisierter Repräsentant	Rappresentante Autorizzato	Representante Autorizado
$\Sigma$	Contains sufficient for <n> tests	Contenu suffisant pour <n> tests	Enthält ausreichend Reagenzien für <n> Tests	Contenuto sufficiente per <n> test	Contiene lo necesario para <n> test
<b>CAL</b>	Calibrator	Calibrateur	Kalibrator	Calibratore	Calibrador
<b>CONJ</b>	Conjugate	Conjugué	Konjugat	Conjugate	Conjugar
<b>MT PLATE</b>	Microtiter plate	Plaque de microtitration	Mikrotiterplatte	Piatto Microtiter	Plato Microtiter
<b>SAMPLEDIL</b>	Sample diluent	Diluant	Proben verdünner	Diluire prova	Diluir muestra
<b>BUF WASH 20X</b>	Wash buffer Concentrate	Solution de lavage Concentré	Waschpuffer Konzentrat	lavari di tampone concentrare	lavado de defensa concentrar
<b>SUBS</b>	Substrate	Substrat	Substrat	Substrato	Substrato
<b>SOLN STOP</b>	Stop solution	Solution d'arrêt	Stop Lösung	terminare la soluzione	Parar la solución